Alliance™ II

Integrated Inflation/Litho Handle

Directions for use	02
Instrucciones de uso	04
Mode d'emploi	06
Bedienungshinweise	08
Istruzioni per l'uso	10
Gebruiksaanwijzing	12
Instruções de utilização	14

Alliance™ II

Integrated Inflation/Litho Handle

Rx ONLY

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.


WARNING

Contents supplied NON STERILE. Do not use if damaged. If damage is found, contact your Boston Scientific representative.

DEVICE DESCRIPTION

The Alliance II Integrated Inflation/Litho Handle is a non-sterile, reusable tool used with the sterile, single use Alliance II Syringe/Gauge Assembly (sold separately), designed to simplify inflation and deflation of balloon dilatation catheters. The Alliance II Integrated Inflation/Litho Handle is also used with the sterile, single-use Trapezoid™ RX Wireguided Retrieval Basket, which is supplied separately. When assembled, the system is intended to aid in the crushing of gallstones during endoscopic retrograde cholangiopancreatography.

The diagram shown below (Figure 1) describes the key components of the Alliance II Integrated Inflation/Litho Handle when used with the Syringe/Gauge Assembly. On the left side of the handle is a blue directional switch that controls the movement of the Syringe/Gauge Assembly plunger or the movement of the basket. Pumping the handle lever inflates and deflates the balloon catheter or extends and retracts the basket to crush the captured stone.

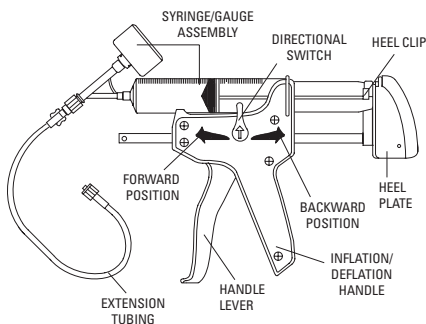SINGLE-USE SYRINGE/GAUGE ASSEMBLY

The Alliance Syringe/Gauge Assembly is a disposable syringe pre-assembled with a disposable pressure gauge. The gauge is graduated in both atmospheres (12 ATM maximum) and kilopascals (1216 kPa maximum). The Syringe/Gauge Assembly is to be used with the reusable Alliance II Integrated Inflation/Litho Handle when performing balloon dilatation.

INTENDED USE / INDICATIONS FOR USE

The Alliance II Integrated Inflation/Litho Handle is indicated for inflation and deflation of balloon dilatation catheters while simultaneously monitoring balloon catheter pressures and for the crushing of biliary calculi when used with the Trapezoid RX Wireguided Retrieval Basket.

PRIOR TO USE FOR DILATATION / LITHOTRIPSY

PRECAUTION: A thorough understanding of the technical principles, clinical applications, and risks associated with balloon dilatation and/or ERCP and mechanical lithotripsy is necessary before using this product.

Inspect the Alliance II Integrated Inflation / Litho Handle for damage and utility. Pump the handle lever to ensure the device's ability to move in the direction indicated by the arrows. The Syringe/Gauge Assembly component of the Alliance II Integrated Inflation/Litho System is supplied EtO sterilized in a sealed tray. Carefully examine the unit to verify that neither the contents nor the sterile package has been damaged during shipment. **DO NOT USE if damaged. Immediately return the damaged product to Boston Scientific.**

INSTRUCTIONS FOR USE FOR DILATATION

1. To insert the Alliance II Syringe/Gauge Assembly into the Alliance II Integrated Inflation/Litho Handle, turn the blue directional switch to the upward position. Align the syringe flanges with the slot in the handle and align the back of the syringe plunger with the slot in the heel plate. Push down on the Syringe/Gauge Assembly firmly to insure proper fit. Snap the heel clip over the syringe plunger. Move the heel plate and syringe plunger all the way forward.

Note: Improper Syringe/Gauge Assembly alignment may cause inability to properly inflate the balloon.

Optional: Extension tubing is provided for your added convenience. If using the extension tubing, connect to the Syringe/Gauge Assembly and the balloon catheter hub using the Luer-Lok®.

2. Turn the blue directional switch backward toward the red arrow. Pump the handle lever to fill the Syringe/Gauge Assembly with 35 ml of sterile water for balloons up to a 20 mm diameter.

Note: NEVER use air or any gas medium to inflate the balloon catheters. Maximum dilatation results are achieved using fluids.

3. Purge excess air from the Syringe/Gauge Assembly by turning the blue directional switch forward toward the green arrow, tilting the assembly so that the air bubbles float towards the Luer-Lok end and pumping the handle lever until all air is purged from the syringe.
4. Attach the balloon catheter hub to the Syringe/Gauge Assembly or to the extension tubing via the Luer-Lok.
5. Pull a vacuum on the balloon catheter by moving the blue directional switch backward toward the red arrow and pump the handle lever until it stops.
6. Remove the protective sleeve from the balloon.

Optional: Apply silicone spray to the balloon to facilitate passage through the endoscope.

Note: DO NOT pre-inflate or pre-test the balloon.

7. To inflate the balloon, move the blue directional switch forward toward the green arrow, holding the Alliance II Integrated Inflation System at a downward angle to prevent air from entering the balloon.
8. Pump the handle lever until the gauge on the Syringe/Gauge Assembly registers pressure.

Note: Refer to the balloon package label for the maximum rated balloon pressure. The gauge will read pressure in both atmospheres (ATMs) and kilopascals (kPa).

9. Continue inflation until the desired pressure is achieved.

PRECAUTION: DO NOT exceed the maximum balloon pressure specified on the balloon.

10. To deflate, turn the blue directional switch backward toward the red arrow and pump the handle lever until it stops.

Note: The blue directional switch may be harder to turn when the Syringe/Gauge Assembly is inflated to high pressures.

PRECAUTION: Slide arm may retract rapidly when used with balloons rated above 5 ATM (507 kPa).

11. Return the blue directional switch to the upward position to detach the balloon catheter.

DIRECTIONS FOR USE WITH LITHOTRIPSY

1. To attach the Trapezoid™ RX Wireguided Retrieval Basket to the Alliance™ II Integrated Inflation/Litho Handle, turn the blue directional switch to the upward position.

2. Align the thumb ring and the handle ring with the heel plate and body portion of the Alliance II Integrated Inflation/Litho Handle respectively.

3. Push the basket handle down into the Alliance II Integrated Inflation/Litho Handle until secure.

Note: The heel clip is not intended for use with the Trapezoid RX Wireguided Retrieval Basket.

4. To retract the basket, turn the blue directional switch forward toward the green arrow. Firmly pump the handle lever until the calculus is fragmented.

Note: Once the calculus is fragmented, stop squeezing the handle of the Alliance II device to avoid disengaging the Trapezoid basket tip.

Note: Basket retraction and extension may be viewed under fluoroscopy.

5. Stop pumping the handle lever when the basket fully retracts into the catheter tip.
6. To extend the basket, turn the blue directional switch backward toward the red arrow. Firmly pump the handle lever until the basket is fully extended.
7. To detach the lithotripsy basket from the Alliance II Integrated Inflation/Litho Handle, return the blue directional switch to the upward position, and pull upward on the lithotripsy basket handle.

CLEANING INSTRUCTIONS

DO NOT immerse the unit in disinfecting solutions. DO NOT steam autoclave.

The Alliance II Integrated Inflation/Litho Handle can be wiped with clean water and disinfecting solution such as 2% glutaraldehyde. Follow the solution manufacturer's recommended instructions for safe handling.

PRECAUTION: Ensure that the Alliance II Integrated Inflation/Litho Handle is dry prior to use, otherwise slippage may occur.

HOW SUPPLIED

The Alliance II Integrated Inflation/Litho Handle is packaged with one non-sterile Alliance II Integrated Inflation/Litho Handle. Do not use if package is opened or damaged. Do not use if labeling is incomplete or illegible.

STORAGE

Store at controlled room temperature. **DO NOT** expose to organic solvents, ionizing radiation or ultraviolet light.

WARRANTY

Boston Scientific Corporation (BSC) warrants that reasonable care has been used in the design and manufacture of this instrument. **This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.** Handling, storage, cleaning and sterilization of this instrument as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond BSC's control directly affect the instrument and the results obtained from its use. BSC's obligation under this warranty is limited to the repair or replacement of this instrument and BSC shall not be liable for any incidental or consequential loss, damage or expense directly or indirectly arising from the use of this instrument. BSC neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this instrument. **BSC assumes no liability with respect to instruments reused, reprocessed or sterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.**

Luer-Lok is a trademark of Becton-Dickinson and Company.

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All rights reserved.